ZAMST FILMISTA WRIST SUPPORT INSTRUCTIONS FOR USE (For left or right side)

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use, and store it away for future reference.

Usage:

Wrist support

Read this first:

- This product can be used for the left or right side.
- Support may not be provided by this product under certain usage conditions.
- This product cannot be used to treat wrist disease, disorder or injury.
- Be sure the product size is correct before use. Refer to the size chart.

When using this product:

- Due to the characteristics of materials used in this product, it may change colors in some storage conditions.
- When storing this product, do not hang excessive weight on the product. Doing so may cause unwanted folding lines, wrinkles, or weaken the hook-and-loop fastening capability.
- Do not leave this product in high-temperature and high-humid areas. Doing so may cause the product to deteriorate or delaminate.
- At times small dimples may appear on the surface of the product or it may become slightly deformed due to the material. The quality is not compromised when this occurs.

Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

⚠ WARNING: Using the product incorrectly may result in serious injury.

⚠ CAUTION: Using the product incorrectly may result in minor injury, or cause damage to property other than this product.

⚠ WARNING

- To prevent abnormalities or worsening of symptoms, those who suffer from the following conditions should refrain from using the product.
 - Those predisposed to allergies or with sensitive skin.
 - Those who have a fracture, cut, numbness, swelling, eczema, rash, or other abnormality in the area to be worn.
- Be sure to follow the rules below and use the product correctly. If not used correctly, it could cause circulatory issues or injury.
 - Do not use this product for any parts other than wrist.
 - Be sure to use the correct size product. Refer to the size chart.
 - Do not use while sleeping.
 - When wearing the product for a long period of time, re-adjust as needed. Constant pressure for a long period of time may cause skin or circulatory issues.
 - Use top/bottom, inside/outside correctly.
 - Follow the application steps for correct use of this product.
 - Do not fasten the main body of the product too tightly.
 - If the product loosens or comes off during use, be sure to refasten it correctly by following steps from the beginning.
 - If this product shows wear, damage or the hook-and-loop fastener no longer works well, do not use it.
 - Do not attempt to modify, or repair this product.
- If any of the following abnormalities occur when using this product, immediately stop using the product, and consult with a physician. Continued use may worsen symptoms.
 - Cases in which you may hurt other people when they come in contact with the product.
 - If this product causes discomfort.
 - If an injury, fracture, cut, pain, numbness, swelling, eczema, rash, etc. occurs in the area the product is worn.

⚠ CAUTION

- Hard materials are used for the hook-and-loop fastener. In some rare cases, fingers or skin may be damaged, so please use the product carefully.
- Sweat and friction may cause this product's color to fade, or transfer to other materials.
- Make sure the hook-and-loop fastener does not attach to clothing. Doing so may cause it to become frayed or damaged.

Care and cleaning

⚠ CAUTION

Do not wash this product with light colored items. By doing so, color may fade or bleed onto other materials.

When washing together with clothing etc., be sure to use a laundry net. If the hook-and-loop fastener attaches to clothing, it may cause them to become frayed or damaged.


- This product can be washed in a washing machine. After washing, hang it dry in the shade.
- When washing this product, attach the hook-and-loop fastener to the main body. Attempting to wash this product without closing the hook-and-loop fastener may weaken its fastening capability.
- Do not use an iron or chlorine bleach. However, oxygen bleach can be used.
- Do not use a dryer, doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning can be done, but avoid tumble drying.
- Do not use alcohol, benzene, etc., doing so may damage the product.

How to apply

- Before attaching this product, carefully read through the "Precautions for use - Be sure to read this". Then follow the steps provided for proper attachment.
- The attachment steps below use the right wrist for reference. The same principles apply to the left wrist as well.


1. Remove all of the hook-and-loop fasteners of the main body of the product, and check over the product's top/bottom and inside/outside. Check the positioning of the product. Strap **A** will be positioned closer to the hand and the logo mark faces the front side.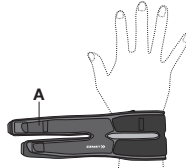


2. Slide the adjustment belt **B**, and pass the hand through the product from the side with the logo mark.


3. Make sure the belt loop **C** is in line with the middle finger, adjust the strap **A**, and close the hook-and-loop fastener **D**. With the hand open, pull the strap **A** to the little finger side (in the case of the left hand, thumb side), make sure it fits tightly on the wrist, then close the hook-and-loop fastener **D**. Be careful not to turn the product at the time.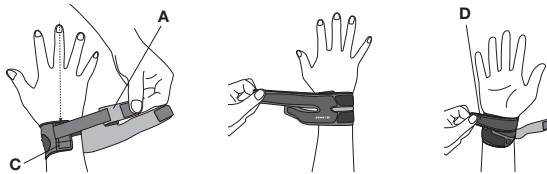

4. Adjust the strap **E** and close the hook-and-loop fastener **F**. Adjust the strap **E** so as not to wrinkle the product, and close the hook-and-loop fastener **F**.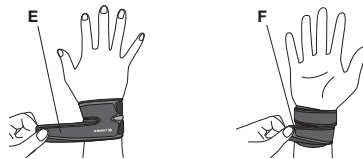

5. Make sure the main body or straps of the product are not loose or improperly attached. If anything looks unusual, start over from the beginning to re-attach the product.

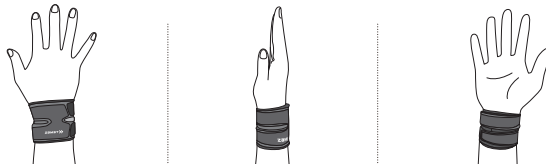Materials:

Fiber: Elastane, Nylon, Polyester
Non-fiber: Polyurethane

* Note: When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.

Disclaimer:

* Product specifications, external view, etc. may be changed without prior notice for improvement purposes.

* All possible measures were taken to ensure product quality, but if a defect, etc. is found, please contact us.

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